

Collection & Shipping Guidelines for Blood Samples (Red/Yellow Top Tube)

START HERE



Cattle



Goat



Sheep



Poultry



Swine

Label each tube
before/after draw with
Animal ID and Tube #

Gently invert tube
5-10x after draw
(don't shake)


Sit samples
upright at room
temp for ~30 mins

Submission form:
Write clearly
(contact info, tests
requested + match
animal ID/Tube #)

Please refrigerate and ship as soon as possible, no more than 3 days post collection




Wrap each tube
with paper towel

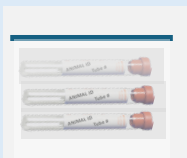

Bundle tubes + seal in
a ziplock bag + wrap
bag with paper towel




Add 1 frozen gel pack
(no dry ice / no frozen water)

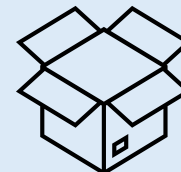

Ship in a box with insulation
(Styrofoam cooler preferred;
no envelopes)

Prepare box as following

Write on box outside:
"EXEMPT ANIMAL SPECIMEN"

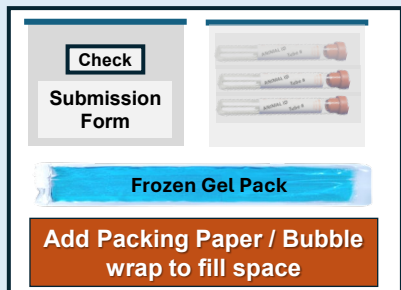

- ✓ Frozen gel pack
- ✓ Ziplock 1: Submission form + payment
- ✓ Ziplock 2: Sealed tubes
- ✓ Avoid loose-fill foam/crumby foam.
Use paper or bubble wrap

- ❖ Ship 1-3-day service
(FedEx/UPS preferred). Avoid USPS
- ❖ In-person drop off at the address
below



SHIPPING

- Samples are received Monday-Friday, excluding holidays

RESULTS

- Be aware of test days to avoid delays
 - Results will be emailed to you

PAYMENT

- We accept card, check, bank ACH, or cash
- If paying by check: include it with submission
- Otherwise: invoice/payment link sent after samples arrive (processing fee included)



Delta Livestock Diagnostics
2815 Exchange BLVD. Suite 100,
Southlake, TX 76092

(870) 232-6401

lab@deltalivestockdiagnostics.com